

Adhesive Primer for Zirconia Bonding

Zirconia Liner *Premium*

Reliable Bond Strength
Easy to Use



 **SUN MEDICAL**
www.sunmedical.co.jp




Zirconia Liner Premium

**A Priming Agent for Zirconia Bonding.
ONE Bottle Type, Just Apply and Dry.**

Reliable Bond Strength

Application of Zirconia Liner Premium on a zirconia surface before bonding with dental adhesive resin cements improves the bond strength and durability.

Tensile Bond Strength to Zirconia* (MPa)

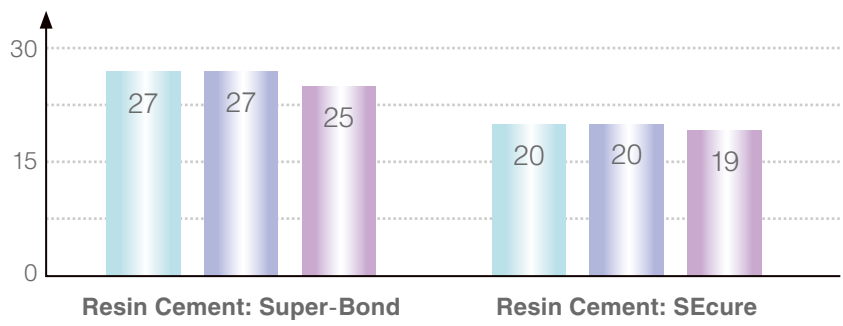

* The zirconia samples are sandblasted
 ■ Immersed in 37°C water for 24 hours
 ■ Thermocycled 5,000 times (5-55°C)
 ■ Thermocycled 10,000 times (5-55°C)

Assures resin cement to bond to zirconia strongly and durably

Clinical Procedure

- ①Dispense 1 or 2 drops of Zirconia Liner Premium to a well of a Plastic Dispensing Dish.
- ②Using an application sponge in the kit, apply the Zirconia Liner Premium to the zirconia surface to be bonded.
- ③Blow lightly if the liquid remains in a dent.

Contents of the kit

- | | |
|----------------------------------|----------------------------------|
| ①Zirconia Liner Premium----- 5mL | ②Plastic Dispensing Dish ----- 5 |
| ③Sponge (L · S) -----1 | |

Made in Japan by



571-2 Furutaka-cho, Moriyama,
 Shiga, 524-0044, Japan
 Phone: +81-77-582-9981 Fax: +81-77-582-9984
<http://www.sunmedical.co.jp>

