MS Coat ONE

Desensitizer for Hypersensitive Teeth

EN

INSTRUCTIONS

IMPORTANT :
READ ALL INSTRUCTIONS THOROUGHLY BEFORE USE.
KEEP THIS LEAFLET AND REFER TO IT PERIODICALLY.

Desensitizer for Hypersensitive Teeth

MS Coat ONE

FOR DENTIST USE ONLY

1. What is MS Coat ONE?

MS Coat ONE is a single-bottle dentin tubule sealant. It reacts chemically with tooth structure and forms a precipitate that blocks dental tubules, to prevent thermal, mechanical and chemical stimulation of odontoblastic processes.

Application of MS Coat ONE is indicated for topical treatment of dental hypersensitivity due to patent dentinal tubules, and will immediately reduce it. Treatment will remain effective for a significant time period. The use of MS Coat ONE will NOT relieve the discomfort of pulpitis.

Indications:

MS Coat ONE is used for the treatment of hypersensitive dentin.

2. Contents of the kit

MS Coat ONE Kit

- ① MS Coat ONE ----- 5mL
- ② Felt Holder ----- 1
- ③ Felt Applicator ----- 100
- ④ Plastic Dispensing Dishes ----- 20
- Attached Documents
 - Instructions

3. Composition

Copolymer with sulfonic acid group, Oxalic acid, Water.

4. Precautions

4.1 Safety

This product contains substances that may cause allergic reactions. Read the following points below.

① Sensitivity

Discontinue use immediately if signs of irritation (rash, etc.) appear. See a physician.

② Avoid contact

MS Coat ONE is an acidic liquid. Avoid contact with mucosa membrane, skin or eyes. Contaminated mucosa membrane or skin should be thoroughly rinsed with running water. If the material enters the eye, rinse immediately with copious amounts of running water and consult an ophthalmologist. Be sure to wear dental gloves.

③ Avoid ingestion

Avoid accidental ingestion while applying and rinsing this material. If ingested, the patient must seek medical attention.

4.2 To get the best results with MS Coat ONE

① Storage conditions

Store MS Coat ONE in a dark place at room temperature (1°C - 30°C). Avoid exposure to direct sunlight and extreme temperatures (above 30°C or below 1°C).

② Avoid Contamination

Once a tooth is sealed with MS Coat ONE, the bond strength of subsequently applied adhesives will be compromised. To seal a surface that will be bonded later, use Hybrid Bond instead of MS Coat ONE.

If a surface treated with MS Coat ONE must be bonded, the surface layer should be removed.

Alternatively, if the surface should be rubbed for 10 seconds with a pledget saturated with Green Activator before bonding with Super-Bond C&B, good bond strengths can be attained.

To avoid contamination, do not mix the components with other materials.

③ Do not reuse

Felt Applicator and Plastic Dispensing Dishes are for one-time use only. Do not reuse and should be discarded after using.

4.3 Disposal

Dispose of empty container or package according to the law about the waste.

As in any dental treatment, the patient's individual constitution and the unique requirement of the clinical case at hand must be considered before selecting materials and conditions for use.

MS Coat ONE should be applied to root-planed surfaces after the period during which re-attachment of gingiva can be expected.

5. How to use MS Coat ONE

① Surface preparation

Tooth surface preparation: Remove plaque and clean the tooth surface with a moistened cotton pledget. Then wipe with a dry cotton pledget.

(Note: If plaque and calculus are severe, preliminary application of MS Coat ONE may be necessary. After scaling, another coat of MS Coat ONE should be applied.)

② Application of MS Coat ONE

a) Dispense 1-3 drops of MS Coat ONE into the plastic dispensing dish.

b) Application of MS Coat ONE: Control moisture with cotton rolls. Apply the liquid with the Felt Applicator for 30 seconds using a pumping/rubbing motion.

Then gently dry the surface with an air syringe for 10 seconds. If the surface remains hypersensitive, repeat the procedure.

③ Rinse

Rinse with water. If the efficacy after treatment is not perfect, repeat the treatment at successive appointments to improve desensitization.

MS Coat ONE

Desensitizer for Hypersensitive Teeth

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MS Coat ONE Kit:

- ①MS Coat ONE ----- 5mL
- ②Felt Holder ----- 1
- ③Felt Applicator ----- 100
- ④Plastic Dispensing Dishes ----- 20
- Attached documents
· Instructions

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Copolymer with sulfonic acid group, Oxalic acid, Water.

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SUN MEDICAL CO.,LTD.